

## Contour FaceSpace (41084) Original FaceSpace (41014)

The FaceSpace is designed to support the weight of a client's head during massage, and is not intended to support full body weight. Therapists are advised to remove the FaceSpace or lower it straight down against the end of the table when leaving the treatment room, if possible. This prevents damage from a client pushing or leaning against it when rising from the table or when getting dressed.

To mount the FaceSpace, release the clamping tension of the Locking Lever and insert the rods into the holes at the end of the table. Adjust the height by lifting the horseshoe platform upward with both hands. The platform pivots up or down to position the angle, as well. Secure these positions by clamping the Locking Lever.

If the FaceSpace droops and will not hold its position under load (or if the Locking Lever is too tight, especially when new), the Tension Nut may have to be adjusted. The Tension Nut is located in line with the Locking Lever at the opposite side (on far left side in photo below). Follow these steps to tighten or loosen the tension.

- Release the clamping tension of the Locking Lever, and remove FaceSpace from the table.
- Turn the horseshoe platform so the underside is facing you (as shown in photo).
- With the rods toward you, hold the Tension Nut with a wrench. You may wish to cover the Tension Nut with a cloth to protect it from being scratched, before using the wrench.
- Grasp the Locking Lever with your other hand and turn it away from you a quarter turn to tighten. (Or turn it towards you to loosen.) If still too loose, repeat these steps to get it to hold, but do not over tighten.
- If the FaceSpace will still not hold its position, it is time to replace it. Face Pillow can be removed and reattached to a new FaceSpace platform, as long as Pillow is in good shape.
- Warranty is one-year from date of invoice.

NOTE: Photo shows Contour FaceSpace with 3/4" rods. Original FaceSpace has 5/16" rods.

